

UVL Series



Out of Sight, Out of Mind

The UVL, or Under Vehicle Lift®, offers an innovative solution to personal mobility. With the lift mounted underneath the vehicle, it remains out of sight and out of your way until needed. This means easy access, a clear side view and maximum interior space – something your friends and family will appreciate on every trip.

Activated by the standard hand-held or optional remote control, the lift glides smoothly from its weathertight aluminum enclosure, ready to gently raise wheelchair users to vehicle floor level. The operation is so simple it will soon become second nature, and the UVL will simply become a dependable part of your daily routine.

Many thoughtful features are included, such as a slip-resistant platform, automatic roll stop and an integrated manual backup system. And because it's so important to you, safety has always been a top priority at Braun. Each lift is subjected to strict quality control procedures and extensive safety testing. The quality construction is further backed by Braun's Three-Year Worry-Free Warranty – the best the industry has to offer.

With the Braun UVL, you'll have confidence and peace of mind knowing that mobility is just the touch of a button away. With over 30 years' experience, you can rest assured Braun has engineered the UVL to fit your active lifestyle.



Unobstructed
Doorway

Threshold Sensor Mat

Inboard Barrier/
Bridge Plate

Weathertight
Storage Cassette

Automatic
Roll Stop

Durable
Powder-Coated
Finish

Ergonomic
Hand-Held
Control





Specifications

Equipment

Hydraulic wheelchair lift for American-made full-size vans (side door)

Lifting Capacity

Up to 600 lbs.

Unit Weight

365 lbs.

Platform Dimensions

30" Usable Width

43" Usable Length

Door Operators

Optional door operators are available for the side doors.

Under-Vehicle Storage


The UVL is stored underneath the vehicle in a weathertight aluminum housing, leaving the doorway completely unobstructed for ambulatory passengers and cargo.

Hydraulic Power Pack

The hydraulic power pack is separate from the lift and is mounted inside the vehicle. A standard manual backup pump allows operation in the event of electrical failure.

Lift Controls

The lift is equipped with a standard hand-held control. For greater convenience, an optional remote control system is available.

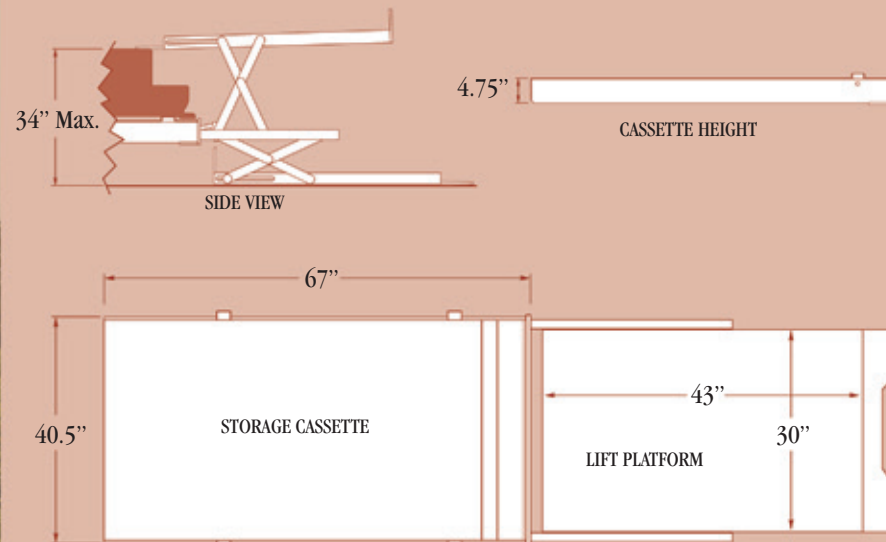

THE NUVL603C IS VA ACCEPTED & FULLY NHTSA COMPLIANT

System Operation

The operator first opens the vehicle's doors manually (if the vehicle is equipped with optional door operators, the doors are opened automatically by pressing the "up" or "down" switch). To bring the lift to ground level, the "down" switch is pressed and held until the platform reaches the ground and the automatic roll stop deploys. After boarding the platform, the operator activates the "up" switch to raise the platform to vehicle floor level. The operator enters the vehicle and then presses the "stow" button to stow the lift in the weathertight compartment. If equipped with door operators, the "door" switch is pressed to close the doors.



UVL® Series Wheelchair Lifts



1-800-THE-LIFT®
For The Dealer Nearest You



INTERNATIONAL
CORPORATE HDQTRS:
P.O. Box 310
Winamac, IN
46996 USA
1-800-843-5438
(574) 946-6153
fax (574) 946-4670

www.braunmobility.com™

32136 © 2005 The Braun Corporation All illustrations, descriptions and specifications in this brochure are based on the latest product information at the time of publication. The Braun Corporation reserves the right to make changes at any time without notice.

